STERILE EO

BIO-SPONGE

COMPRESSAS NEUROCIRÚRGICAS

BIO-SPONGE

NEUROSURGICAL COMPRESSES

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PRESENTATION

Contain ten (10) neurosurgical compresses in each pack wrapped in double blister, properly sterilized.

In the external labels there are information referring the period of validity and production date.

SIZES

REF	WIDTH	REF	WIDTH
2030-1313	13x13mm	2030-1919	19x19mm
2030-1325	13x25mm	2030-2525	25x25mm
2030-1338	13x38mm	2030-2576	25x76mm
2030-1370	13x70mm	2030-3838	38x38mm

Special Measures Under Consult

DESCRIPTION

The Neurosurgical Compresses BIO-SPONGE is assembled in fiber of polyester of extreme purity by a special process of interlacement.

A coating thread with barium sulfate is used to facilitate the post-surgical count.

INDICATIONS

Bio-SPONGE are compresses suitable to be used for tissue protection, including cerebral and other tissues of the central nervous system, during the surgery.

WARNINGS

Do not use this product in case the pack is damage and/or period of validity expired. Open aseptically the pack after certify the

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correct dimension.

Qualified people should only manipulate this product.

INSTRUCTIONS OF USE AND PRECAUTIONS

Use procedures inserted in the technical norms for neurosurgery, using asepsis always when transferring the compresses to the sterile field.

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CONTRAINDICATIONS

This product has specific purpose: neurosurgery. Therefore, is contra indicated for any other purpose.

UNDESIRABLE EFFECTS

Before closing the surgical field, certified that were removed all the neurosurgical compresses For safety, use X-Ray as aid in this procedure.

WARRANTY

We guaranteed the quality of this product since the user follow the instructions of use contained in this label
Biomechanica will not take the responsibility for current accidents of the bad use of this product.

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